



medemagroup



User Manual Mini Crossover X1 Cabin

Serial number: _____ - _____ - _____

Delivery date: _____ Year 20_____

This vehicle was supplied by:

Date: /

Medema A/S

Enggårdvej 7
Snebjerg
DK-7400 Herning
Denmark

Telefon: 70 10 17 55
Mail: info@medema.com

Contents

Intended occupant.....	5
Intended Environment.....	5
Help for the visually impaired	5
Symbols.....	6
Warning!	6
FSN (Field Safety Notice).....	6
Contagion!.....	7
Serious incident.....	7
It is not permitted to use the Mini Crosser for:.....	7
Introduction.....	8
EC - Declaration of conformity.....	9
Medema A/S Warranty.....	10
Mini Crosser X1 Cabin	11
Control panel	12
Preparations / Adjustments prior to use	15
Electric seat adjustment	17
Seat rotation.....	18
Steering column	19
Serial number.....	20
Driving the Mini Crosser X1 Cabin.....	21
Lap belt.....	24
Electromagnetic Compatibility.....	25
When driving in traffic.....	25
Specific driving situations.....	26
Long hills/difficult ground	28
Anti-tilt wheels / stabilisers	29
General care and maintenance	30
Service.....	31
Cleaning	31
Insurance.....	31
Batteries	32
Storage/Charging	33
Battery disposal.....	35
Fuel for heater.....	36
Heater	36
Windscreen wiper / washer	39
Replacing the wiper blade	40
Replacing the headlight	40
Changing the wheels.....	42
Fuses	44
Brakes.....	46
Disengagement.....	47
Securing to vehicle floor with belts	48
Transporting by plane	50

Towing	50
Dimensioned drawing, Mini Crosser X1 Cabin	51
Introduction to the Ergo2 seat	52
Flame resistance	56
Basket on the back of the cabin.....	57
Troubleshooting	58
Electronic faults.....	59
Programming	60
Technical data.....	61
Cassation	63
Weight of the seats.....	64
Changes/adjustments	65
Own notes	66

Intended occupant

The intended occupant of the Mini Crosser X is a person with limited ability of walking themselves.

The occupant must be able to see in order to register traffic signals and other road users when driving in traffic. The occupant can be deaf and/or speechless.

The occupant must have a cognitive ability to understand the operation of a scooter and its buttons and icons.

The Mini Crosser can be driven with only one hand either left or right.

The maximum occupant weight is 150 kg.

Intended Environment

Your Mini Crosser is designed to provide optimal stability under normal riding conditions – dry, level surfaces consisting of concrete, asphalt, packed soil, grass and gravel.

The Mini Crosser is a very stable vehicle, BUT it can tip over if weight is incorrectly distributed and driven irresponsibly.

See the “Specific Riding Situations” section on page 27.

Operating temperature: -25° to +50° Celsius

Storage temperature: -40° to +65° Celsius

Do not use the Mini Crosser in fresh or salt water.

Help for the visually impaired

If you have difficulty reading small print in the user manual, we recommend that you visit our website, where you can read this manual in PDF format. You can enlarge the PDF manual on your PC monitor to suit your needs and preferences.

If you find it difficult to understand the manual and have general questions about the product, please feel free to contact us. You can find our contact info on page two in this user manual.

You can find manuals for all our products on our web page www.medema.dk. Or contact Medema A/S, and we can send the manuals in a mail for you. Find the contact information on page two in this manual.

FSN (Field Safety Notice)

All information concerning safety can be found at www.medema.com, which is always updated with the latest safety information. In the event of important safety-related changes, we will notify our customers directly (FSN).

Symbols



Used in the manual to indicate sections describing situations where extra care is required owing to the risk of personal injury.



Used to indicate sections on electromagnetic compatibility (EMC).

Warning!



For safety reasons the vehicle must not be lent to persons who are not completely familiar with it. The vehicle is designed for one person only.



The Mini Crosser X1 Cabin has been designed for users weighing max. 150 kg.

Contagion!

A standard Mini Crosser is equipped with tyres that does not contage, but if another type of tyres are used, it can sometimes rub off on floor coverings, particularly linoleum. Medema A/S assumes no responsibility in case of contagion.

To prevent this, we recommend that you protect delicate floors with some sort of driving surface.

Serious incident



Any serious incident that occurs in connection with Mini Crosser must be reported to the manufacturer and the competent authority in the region of the country where the user and/or patient resides.

It is not permitted to use the Mini Crosser for:

- transporting more than one person at a time.
- transporting heavy objects that exceed the maximum user weight of the Mini Crosser.
- towing or attaching anything to the rear bumper.

Introduction

Congratulations on your new Mini Crosser X1 Cabin electric mobility Mini Crosser.

You have now taken possession of an electric mobility Mini Crosser developed for outdoor driving by active users. It is what is called a Class C vehicle in according to the European standard EN 12184.

For optimum enjoyment of this vehicle - and to avoid breakdowns and accidents - we recommend that you read this User Manual carefully. As a new user, you should pay particular attention to the section entitled "Driving the Mini Crosser X1 Cabin".

The Mini Crosser X1 Cabin is designed for safe travel for at least 10 years, providing it has service and safety inspections every third year. The service must be carried out by an authorized workshop.



IMPORTANT! For safety reasons it is of the utmost importance that service and safety check intervals are complied with, as this minimises the risk of brake failure and short-circuits in the wiring, which could generate heat and cause a fire.

We offer a wide range of accessories for the Mini Crosser X1 Cabin that can make everyday life easier for you. You are always welcome to contact us for further information on special accessories and adaptations.

Medema A/S is not responsible for any damage or injuries caused by inappropriate or unsafe use of the Mini Crosser X1 Cabin.

If you have any further questions about the Mini Crosser X1 Cabin or this User Manual, you are always welcome to get in touch. Our contact details are as follows:

Medema A/S

Tel: +45 7010 1755

Email: info@minicrosser.com

Internet: www.minicrosser.com

NB: Errors and omissions excepted. We reserve the right to update this manual as required.

Manufacturer
 Medema A/S
 Enggårdvej 7
 DK-7400 Herning

Herning, 23, March 2023

EC – Declaration of conformity Mini Crosser model-X1

Medema-A/S declares on our sole responsibility that the product range mentioned below are manufactured in accordance with our incorporated standards and are in conformity with the following directives and standards if they are used according to our instructions:

Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

EN 12184:2014 Electrically powered wheelchairs, scooters, and their chargers - Requirements and test methods.

Directive 2006/42/EC of the European Parliament and of the Council of 17 May 2006 on machinery, and amending Directive 95/16/EC.

Directive 2011/65/EU of the European Parliament and of the council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

DS/EN IEC 63000:2018 Technical documentation for the assessment of electrical and electronic products with respect to the restriction of hazardous substances.

ISO 14971:2019 Medical devices – Application of risk management to medical devices.

Type of use : Transporting walking disabled people.
 Risk Class : I according to rule 1 of EU MDR 2017/745.

The product range in the **Mini Crosser model-X1** includes the following items

Product name	Basic UDI-DI
Mini Crosser X1 3W Black metallic 10 km/h	5707299MCX175
Mini Crosser X1 3W Black metallic 13 km/h	
Mini Crosser X1 3W Black metallic 15 km/h	
Mini Crosser X1 3W Copper orange metallic 10 km/h	
Mini Crosser X1 3W Copper orange metallic 13 km/h	
Mini Crosser X1 3W Copper orange metallic 15 km/h	
Mini Crosser X1 4W Black metallic 10 km/h	
Mini Crosser X1 4W Black metallic 13 km/h	
Mini Crosser X1 4W Black metallic 15 km/h	
Mini Crosser X1 4W Copper orange metallic 10 km/h	
Mini Crosser X1 4W Copper orange metallic 13 km/h	
Mini Crosser X1 4W Copper orange metallic 15 km/h	
Mini Crosser X1 HD 4W Black metallic 10 km/h	
Mini Crosser X1 HD 4W Copper orange metallic 10 km/h	
Mini Crosser X1 CAB Black metallic 10 km/h	
Mini Crosser X1 CAB Black metallic 13 km/h	
Mini Crosser X1 CAB Black metallic 15 km/h	
Mini Crosser X1 CAB Copper orange metallic 10 km/h	
Mini Crosser X1 CAB Copper orange metallic 13 km/h	
Mini Crosser X1 CAB Copper orange metallic 15 km/h	

Medema-A/S



Klaus Smedegaard
 General Manager

Document No. 202337 Rev. No. 7

Medema A/S

Enggårdvej 7
 Snejbjerg
 7400 Herning

CVR/VAT: DK 51 89 27 12
 SRN: DK-MF-00000981

Phone: 70 10 17 55

info@medema.com
 www.medema.com



medemagruppen

Medema A/S Warranty

1. There is a 2-year warranty, with the exception of worn parts such as tyres, hoses, fuses, light bulbs, bushings and brake pads.
2. If you want to make a claim for a part under the warranty, it must be intact. The warranty will be void if the product has been removed or appears to have been mishandled.
3. There is a 5-year warranty against breakage and corrosion of the undercarriage.
4. For the warranty to cover batteries, the charger used must be sold by Medema A/S.
The fabric, type and serial number must be indicated on the warranty specification, otherwise, warranty coverage may be denied.
5. Items returned under a warranty claim must be suitably packaged to prevent damage during transport. Items damaged during transport due to poor packaging will not be covered by the warranty.
6. The warranty does not cover freight costs.
7. The warranty requires annual inspection and maintenance of the Mini Crosser by authorized professionals.
8. **Warranty on Batteries and Chargers**

Mini Crosser scooters are delivered without batteries and chargers in some countries.

Information regarding warranty and service for these products must therefore be obtained directly from the dealer who supplied the batteries and charger.

Please contact your local Mini Crosser dealer for further information.

Mini Crosser X1 Cabin

The Mini Crosser X1 with a cabin is characterized by being spacious, quiet and offering good visibility through the large windows.

It can be equipped with a reversing camera (accessory), which makes it easier to drive backwards. The side windows can be opened to let in fresh air, and in the summer it is also possible to remove the doors.

The cabin keeps you warm and dry all year round. Equipped with a diesel heater, you can even go out on a cold winter day.



The Cabin below is shown with accessories.



Control panel

Avoid subjecting the control panel to jarring and knocks. Take care when driving and avoid hitting obstacles.

The manufacturer accepts no liability in respect of the unauthorised opening, adjustment or modification of the control panel.

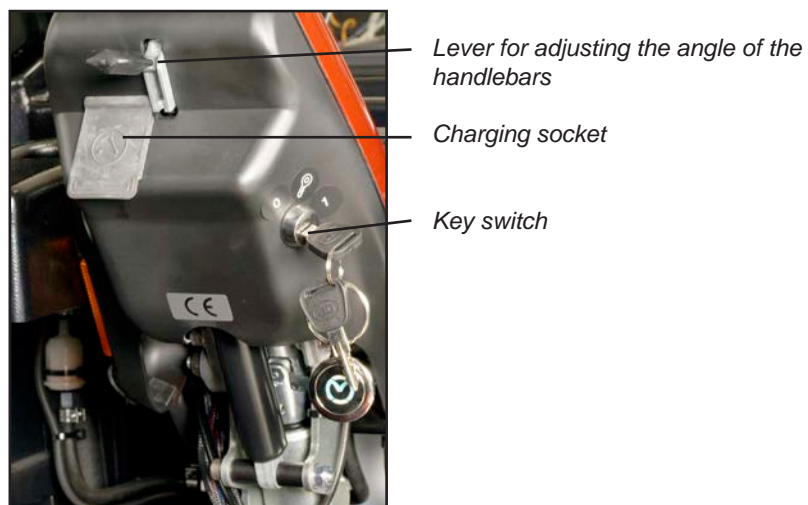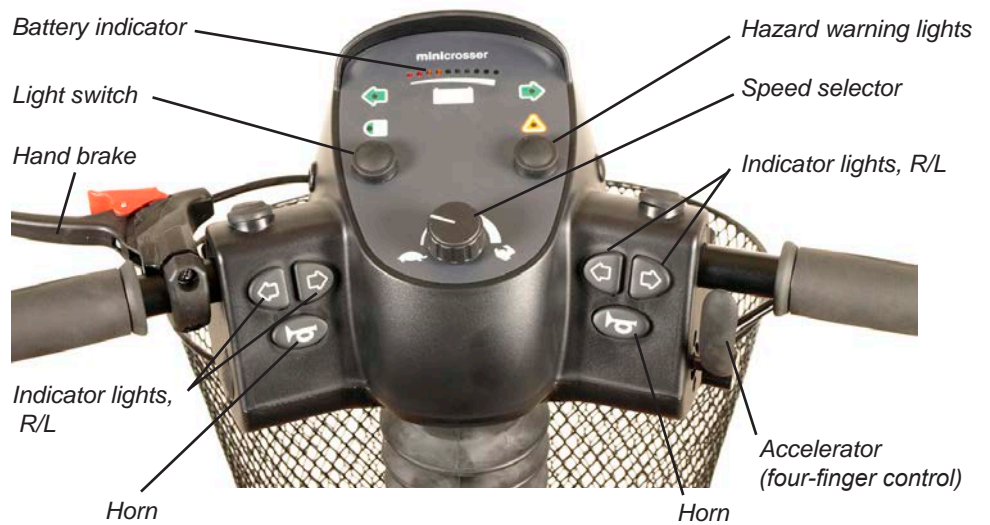



Air channel
Windscreen wiper + washer
Heat regulation



Accelerator (four-finger control)
Accelerator (four-finger control)
The accelerator forward and backwards are physically connected, which means that when one or the other is activated, the other will move the opposite way.


Designation	Description
Key switch	To start the vehicle: Turn the key to driving position (1) When the Mini Crosser is turned on, the control system performs a safety check of the electrical system. The check lasts half a second. If the accelerator is operated during this time, the Mini Crosser will not be able to move until you have turned the key to 0 and back to 1.
Speed selector	The tortoise in minimum position indicates the lowest speed range. 0-6 km/h The hare in maximum position indicates the highest speed range. Up to 15 km/h
Battery indicator	Lights up when the key is turned. Indicates, after half a second, the battery charge level. It gives a more precise indication after approx. 1 minute of driving. When the red, amber and green sections are lit, the batteries are fully charged. When the indicator is at the bottom of the amber section, the batteries should be charged as soon as possible. If only the red section is lit or flashing, the batteries must be charged immediately.
Hand brake	Acts on the rear wheel and should only be used as an extra /emergency brake and parking brake. It can be secured in locked position.
Accelerator	Activate the arm at the front slowly and the Mini Crosser will start to move forwards. The more the arm is pressed, the faster the Mini Crosser will move. When the arm is released, it will flip back to starting position of its own accord and the Mini Crosser will stop. Activating the rear arm will cause the Mini Crosser to reverse. The magnetic brake on the rear wheels will engage when the Mini Crosser is stationary and is equivalent to the parking brake on a car. The accelerator can also be used to control speed downhill. In this case the motor will act as a brake.
Light switch	Turns on the front and rear lights. The key switch must be in position 1 (driving position).
Hazard warning lights	Turns on all the indicator lights at once. Also works when the key is removed or turned to position 0 (stop position).
Indicator switch	Left arrow: indicator, left-hand side. Right arrow: indicator, right-hand side. Activate the same switch to turn the indicators off.
Horn	Powerful electric horn. Press the horn switch to activate.
Charging socket	Turn off the key switch during charging. Please note that it is not possible to drive the Mini Crosser while the batteries are charging. See also the section entitled "Batteries and charging".
Windscreen wiper	Starts the windscreen wiper. Notice the variable speed adjustment.
windscreen washer	Spraying washer fluid on the front window.
Fresh-air/hot-air switch	Switch to toggle between heated and unheated air.
Heat regulation	Adjustable heat in the cabin. Note: Heat can be turned on even if the key is not in the key switch. Heating ribs should NOT be covered. REMEMBER! turn off the heat when you leave the vehicle. Otherwise the batteries may be flat the next time you want to use it.

Preparations / Adjustments prior to use

Adjusting the height of the seat


Lift both armrests up, and fold the seatback. Release the handle so the seat can turn. Turn a little and lift the seat off.

Make sure you keep your back straight when lifting the seat, which is very heavy. (See below.)

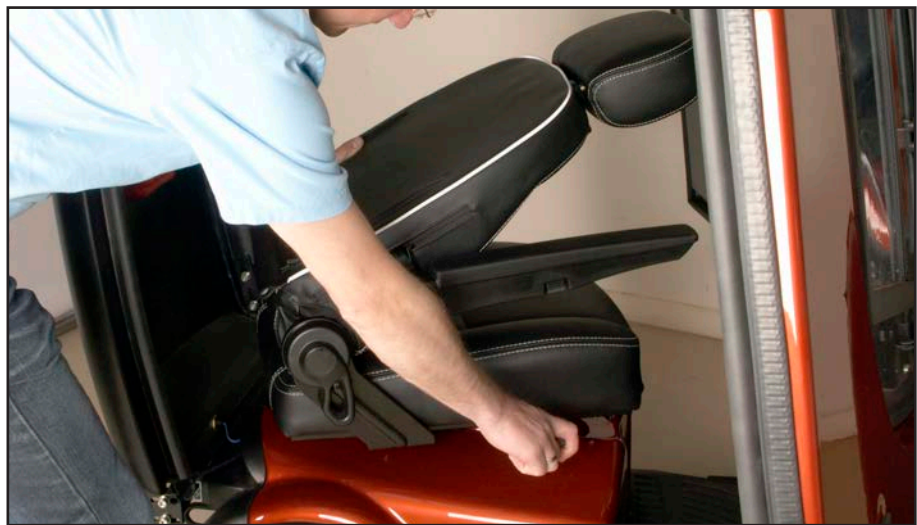

Lift the seat off the seat tube.


Height adjustment seat post standard

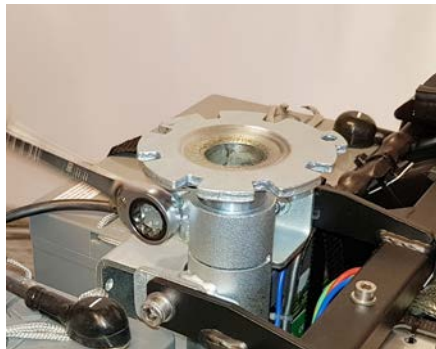

*Loosen the counter nut.
Use spanner, 17 mm.*



Adjust the seat post to the desired position. The seat post is marked with a ring at each centimetre.

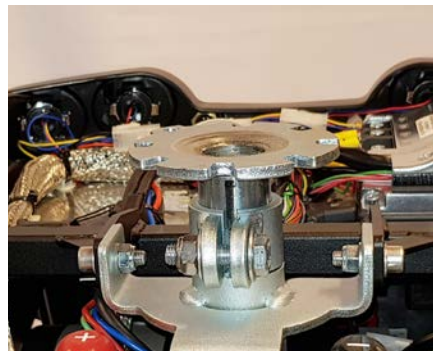


Be aware that the black line is holding its position after movement, so the seat is straight in front of the steering.




*Tighten the screw, here after the counter nut.
Mount the seat. Adjust the lining if necessary.*


Height adjustment seat post with position bolt (option)

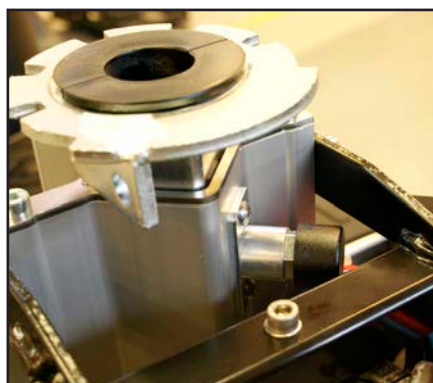

The release button pressed in. Press the button out to release the seat tube.

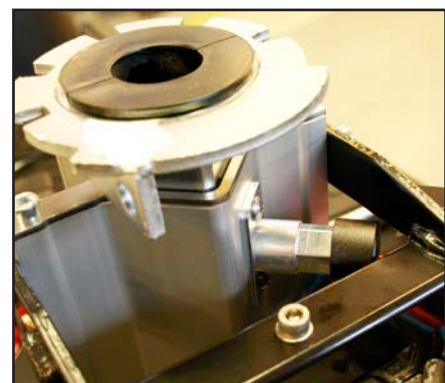

Release button pressed out. Now the seat tube can be moved up/down to the height you wish. Press the release button again and make sure it is fitted in one of the 5 holes in the seat tube.

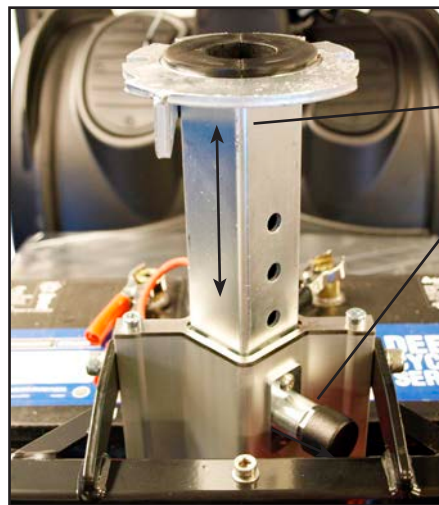

Seat tube

Release

Seat tube in top position. There are five holes for adjusting the seat tube. The release must be in a hole for the seat to be stable.

Electric seat adjustment

The Mini Crosser can be fitted with electric seat adjustment as an optional extra.

To raise the seat, press the switch up. To lower the seat, press the switch down. If the switch is released, the seat will stop automatically. (See figure below.)



For safety reasons, a switch has been incorporated that reduces the speed by 50% when the seat is raised more than 7.5 cm.



Switch for electric seat adjustment

Electric seat adjustment. If the switch is pressed up, the seat is raised. If the switch is pressed down, the seat is lowered.


IMPORTANT!



The Mini Crosser is most stable when the seat is in its lowest position.



Take extra care when lowering the seat on a Mini Crosser using electric seat adjustment. Make sure that nothing is trapped in the space between the seat and the chassis.

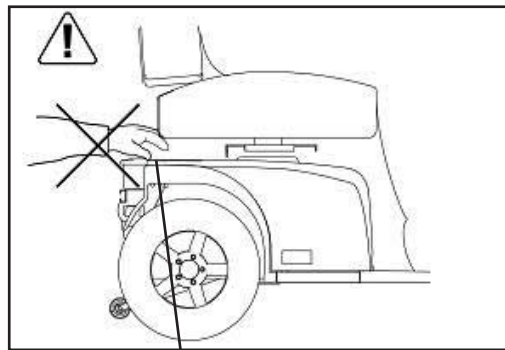

Avoid trapping anything when lowering the seat.

Seat rotation

It is only possible to rotate the seat in a Mini Crosser with Cabin if both doors are open.

Pull the release lever back. The seat can be rotated 90° to either side. When the lever is released, it engages with the seat and holds it in place at 45° intervals.

Other seats that can be supplied for the Mini Crosser work on similar principles. The release lever is normally mounted on the right, but can be put on the left if so wished.



Release lever for seat rotation.

Release lever for seat forward/back.

Steering column

Can be adjusted forward/back with the release lever. Pull down on the lever and pull the handlebars closer for a good driving position. Use the lever again and push the steering column forward to make it easier to get out.

The height of the steering column can be adjusted approx. 11 cm up/down. First remove the rubber plug covering the Allen screw. Adjust the height using a 4 mm Allen key.



Adjusting the height of the handlebars.

Lever for adjusting the angle of the handlebars




Adjusting the height of the handlebars.

Allen screw for height adjustment

Serial number

All vehicles have a serial number plate showing the year of production, month and serial number. The same serial number can also be found on the front of the User Manual.

Please quote this number when making inquiries about servicing, spare parts, etc.

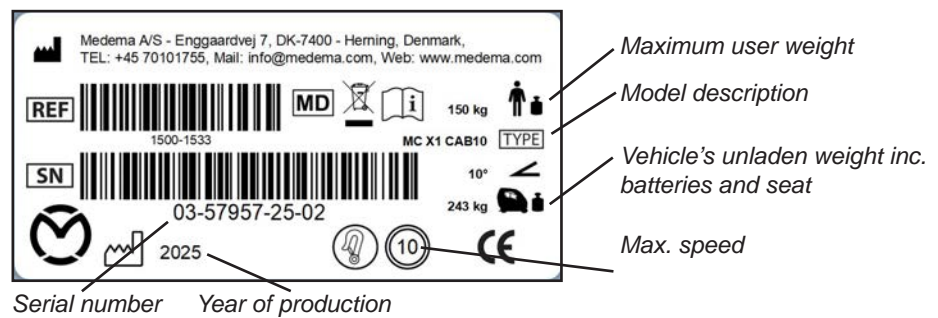

Position of serial number plate on vehicle.

Driving the Mini Crosser X1 Cabin

Getting in and out

It is important to learn a safe technique for getting in and out of the Mini Crosser.

In general the following method should be followed:

- Make sure that the Mini Crosser X1 Cabin is off (switch turned to 0) when getting in and out, otherwise the Mini Crosser could start if the accelerator was touched inadvertently.
- Make sure that the brake is on. (Lever for disengaging motor in top position.)
- Put the steering column in vertical position.
- If necessary, raise the armrest.

Note! - Do NOT use the dors as support, when entering or getting out of the vehicle.

For some users the assistance of an attendant can be recommended. The attendant should:

- Take care not to injure him/herself when lifting/lowering/supporting the user.
- Make sure that the Mini Crosser is stable and unable to move. Turn off the Mini Crosser and check that the brake is on.
- Make sure that the seat the user is being moved to is stable.



IMPORTANT!

The Mini Crosser will turn itself off automatically after it has been stationary for 10 minutes even if the key has been turned to driving position. The battery indicator flashes with 6 lamps at a rate of about 3 seconds.

To restart the vehicle, the key must be turned to stop position (0) and then to driving position (1).



Turn the Mini Crosser off, return the steering column to upright position and raise the armrest.

General safety advice

Make sure that the backrest is upright and the seat is as low as possible.

Positioning belts are recommended if the user is unable to maintain a good driving posture independently.

Adjust your driving to road conditions. Take account of light, traffic and weather. Be particularly careful when driving in the dark or in bad weather, such as rain or snow. Avoid driving on gradients with poor surfaces, such as: snow, ice, new-mown grass, wet grass and wet leaves.



Never drive when under the influence. This applies not only to alcohol, but also to drugs and medicines.

Reduce speed immediately if you feel you are losing control.

Always use the indicators when changing direction.

Check that lights and indicators are working before driving off. Use your lights when driving after lighting-up time.

As far as possible, grip the handlebars firmly with both hands.



Take care not to have anything in the basket at the front that might inadvertently operate the accelerator.



WARNING! Do not lower the handlebars so far that the accelerator can hit you on the leg when you turn the vehicle and so be operated inadvertently. If the handlebars need to be very low, a twist grip accelerator is recommended. This is particularly relevant if leg supports are being used.



ALWAYS switch the Mini Crosser off when it is not in use. Turn the key to 0.



Warning!

If the vehicle behaves abnormally in any way - makes noises, smells burnt, etc. - stop the vehicle immediately and seek technical assistance. By continuing to drive, there may be danger of fire or that vital parts are destroyed.

Driving

Even though the Mini Crosser is very stable, it can tip over. Avoid sudden changes of speed and direction when travelling at high speed, on poor surfaces and, not least, on slopes. So be extra careful in following the driving tips given below.

Anti-tilt wheels are recommended for driving in very hilly terrain. (Optional extra)

New users are urged to practise the following in an area where there is no other traffic:

- Set the Mini Crosser to low speed. Drive forwards and backwards. Gradually turn the speed selector up and feel the change in the speed of the Mini Crosser.
- Practise starting and braking. Get used to the Mini Crosser's response time.
- Practise driving in a narrow space, similar to inside a shop or through a door.
- Practise turning, and get a sense of how much space is required. Always drive slowly when turning. Practise reversing too.
- Practise cornering and driving over obstacles and on slopes. Always drive straight up/down kerbs and ramps. Never at an angle. See the illustrations on the following pages.
- Try braking at different speeds and notice the stopping distances.
- If possible, try driving on different surfaces (road, grass and gravel).
- Practise assessing how far you can drive on a single battery charge. Note how quickly the battery indicator changes from green to amber to red.



Note!

The driving distance of the Mini Crosser will be reduced when driving in hilly districts, into a head wind, in cold weather and with low tyre pressure.

Traffic regulations

The traffic legislation for Mini Crossers varies from country to country. Before starting to use the vehicle outdoors, it is the user's responsibility to familiarise him/herself with the relevant legislation.

Lap belt



Lap belt

If there is a need for it you can, as an accessory, mount a lap belt or harness belt on Mini Crosser X.

The belt is designed to stay safe to use for at least 10 years.

Electromagnetic Compatibility

The Mini Crosser meets the requirements for use of an electric scooter in an environment with electromagnetic noise.

Warning!



In rare cases, electromagnetic noise may affect the performance of the Mini Crosser. Sources of such noise include radio and TV stations, amateur radio transmitters, electrical generators or high-power sources.

If such equipment is used nearby, it is recommended that you switch off the Mini Crosser.



If the Mini Crosser starts to move involuntarily, or if the brakes are released, switch off the Mini Crosser as soon as it is safe to do so.

Warning!



In rare cases, a Mini Crosser may interfere with the operation of devices in the environment that emit electromagnetic fields, such as alarm systems in shops, automatic doors, etc.

When driving in traffic...


Be particularly aware of the following when driving in traffic:

- The Mini Crosser is a low vehicle and not always easy for other road users to see. Make quite sure that other road users have seen you before driving onto the highway.
- Keep an eye on traffic behind you. Keep well over to the side of the road when driving on busy roads.
- Turning right and left at crossroads. Be aware of cyclists and pedestrians. Follow the rules of the road for cyclists.
- How quickly things are happening. How long do the lights stay green? How quickly are cars approaching? etc.

Specific driving situations


Up kerbs

- Stop at right angles to the kerbstone about 5-10 cm away from it. Keep an eye on other road users.
- Lean forwards.
- Accelerate moderately so that your vehicle can overcome the obstacle. Do not stop halfway, but reduce speed once the vehicle is up.
- If the kerb is too high, do not try again, but find an alternative route.

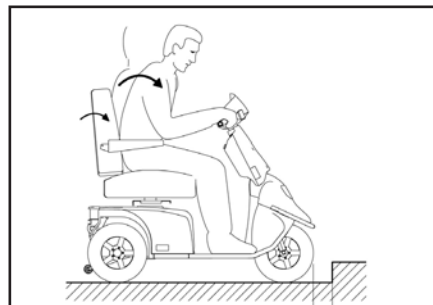



Down kerbs

- Lean back.
- If you are driving onto a road with traffic, keep an eye on other road users.
- Drive forwards and down the kerb at low speed. Make sure that your anti-tilt wheels (if fitted) do not catch on the edge.

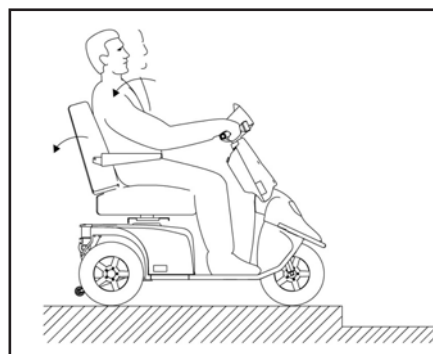



Up a ramp/hill

- There is a risk of tipping over backwards if the seat is pushed back when you start driving up a slope, e.g. a ramp.
- Pull the seat forward! Check that any ramp is stable.
- Lean forwards.
- Accelerate moderately so that your vehicle can overcome the obstacle. Do not stop halfway. Reduce speed once the vehicle is up. If you need to start on a hill, accelerate slowly so as not to tip over backwards.
- Avoid many starts / stops when driving on steep hills. It can over-heat the Mini Crosser.

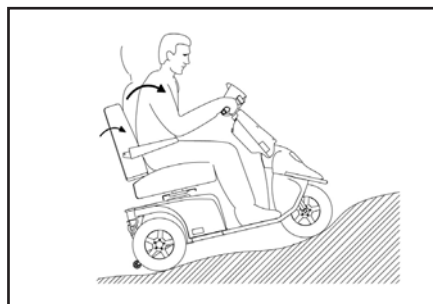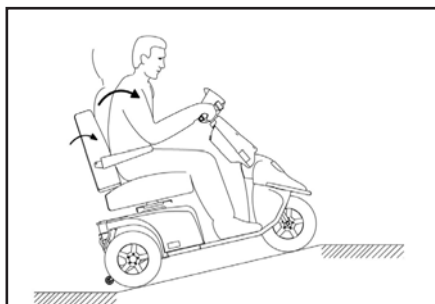

Down a ramp/hill

- Check that any ramp is stable.
- Lean back.
- Drive slowly down. Avoid stopping midway on short, steep slopes. On long hills, it is advisable to stop every now and again if you feel your speed is getting too high.

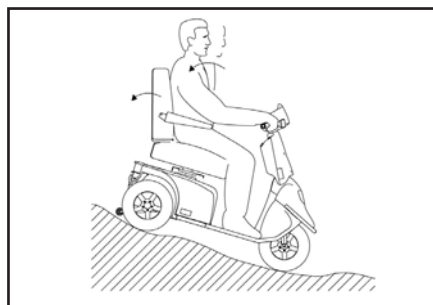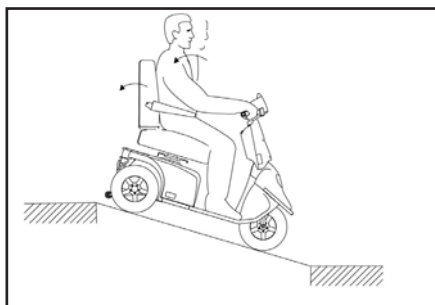

Across a slope

- Lean into the slope.
- Avoid sudden and sharp movements. This is particularly relevant when reversing.
- Always drive at low speed.

Long hills/difficult ground

The Mini Crosser X1 Cabin is at risk of overheating on very long and steep hills or on very soft ground, possibly also combined with high user weight.

To protect the Mini Crosser Xotor from overheating, there is a built-in thermoswitch, which initially sets the Mini Crosser to half speed.

However, if you carry on moving over difficult ground, the Mini Crosser will at some stage stop completely. The Mini Crosser must be left to cool down for 3-5 minutes before it can start again.

Important!



Avoid many starts / stops when driving on steep hills. It can overheat the Mini Crosser.

To reset the fault, switch off the Mini Crosser then turn it on again.

If the Mini Crosser has not had long enough to cool down when it is restarted, it will initially only run at half speed.

Note:



The situation described above will never happen in normal use. It only happens if the conditions are extremely challenging.

Important!



Braking distances on slopes can be significantly longer than on flat terrain.

Anti-tilt wheels / stabilisers

The Mini Crosser is a very stable vehicle. HOWEVER, in the case of incorrect weight distribution or careless driving there is a risk of tipping over.

We therefore recommend fitting anti-tilt wheels in such circumstances. (See picture below). Contact Medema for more info. Find contact information on page two in this manual.



Anti-tilt wheels

General care and maintenance

A Mini Crosser X1 Cabin does not require much maintenance. It should be kept in a generally good condition, however. The following should be checked regularly:

- Tyre pressure (if pneumatic tyres are fitted)
- Tyre wear
- Keep the control panel, the charging socket and the electronics box under the seat dry.
- Battery charging



Never wash the Mini Crosser with a high-pressure cleaner or direct water jet! This could damage the Mini Crosser's electronics.

To keep the Mini Crosser in good condition safety-wise, we recommend the following regular checks:

Daily:

- Test the indicators and driving lights before using the Mini Crosser in the dark or poor visibility.

Every three months:

Test the brakes and motor disengagement

With the disengagement lever up, it must not be possible to push the Mini Crosser.

Test the brake disengagement function

When the brake disengagement lever is down, the battery indicator should flash to show an error if the Mini Crosser is turned on. In this case the Mini Crosser X must not be able to move when the accelerator is activated.

Test the hand brake.

Apply the hand brake for a couple of seconds at low speed. This will ensure that the lever arm and brake shoes do not seize up.

Lubricate the lever arm on the brake hub with acid-free oil - left-hand rear wheel.

Test the windscreen wiper.

Spray washer fluid on the window and check that the wiper blade works properly.

Service

The Mini Crosser X1 Cabin is designed for safe travel for at least 10 years, providing it has service and safety inspections every third year. The service must be carried out by an authorized workshop.



IMPORTANT! For safety reasons it is of the utmost importance that the servicing and safety check intervals are complied with, as this minimises the risk of brake failure and short-circuits in the wiring, which could generate heat and cause a fire.

(For further information see the Service Manual.)

Cleaning

Clean the Mini Crosser with a damp cloth. If necessary, wipe dry with a chamois leather.

Wipe the covers dry and polish with car wax.



IMPORTANT! Never use high-pressure cleaner or hose, as it could damage the Mini Crosser's electronics.



Never use solvents such as turpentine, cellulose thinner or similar to clean the vehicle.

Insurance

In the eyes of the law a Mini Crosser X1 Cabin with a maximum speed of 15 km/h is a cycle, so separate insurance is not required.

Most contents/home insurance policies include third-party liability insurance for cyclists and so also cover Mini Crosser X1 Cabin users.

We recommend that you talk to your insurance company about this when the vehicle is delivered. If necessary, comprehensive insurance will have to be taken out separately.

Batteries


The Mini Crosser uses sealed, maintenance-free GEL or AGM batteries.

They do not generate gas and do not have to be topped up with water.

Battery weight

90 Ah = 26 kg

It is important to fit the batteries correctly. The battery terminals are marked +/- . They must be fitted as shown in the sketch below. Make sure that the terminals are properly tightened. For the same reason there must not be any burrs on the cable terminals.

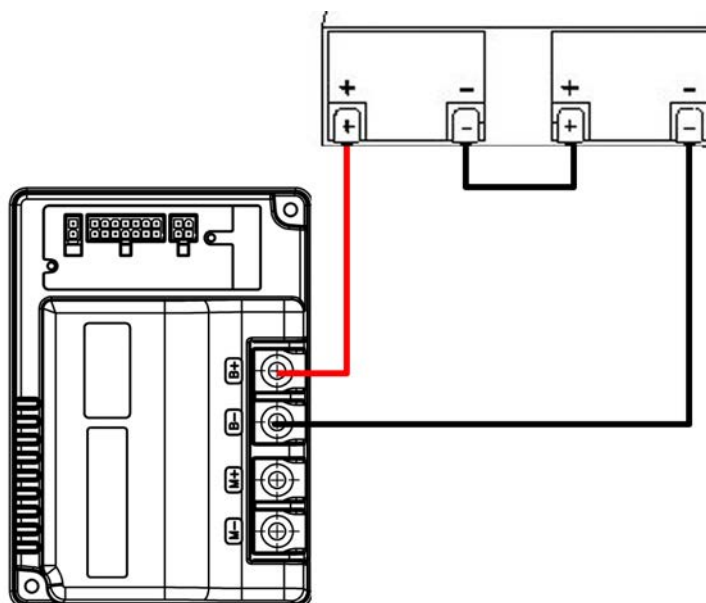

Be aware that the Mini Crosser can vary types of chargers. Contact your dealer for more information about the different types.



NEVER use a charger that are not intended for charging dense, maintenance-free batteries.



Do NEVER use other chargers than the one from the factory without contacting your dealer first.

Storage/Charging

The Mini Crosser is designed for use in all types of weather. However, storage and charging should take place under a roof, alternatively under tarpaulin.

The batteries are best charged at +10 to + 30 °. At colder temperatures, capacity decreases, and thus driving distance.

It is recommended that it charge indoors once in a while.

Note!



The charger must be kept dry, but must not be covered directly over during use.

If you do not use the Mini Crosser for a long time, e.g. for winter storage, you can choose to remove one of the pole shoes for the fully charged batteries. Thereby, there will be no consumption of standby power.

The tires are protected by raising the scooter.

It is also recommended to cover the scooter against dust, rain and sunlight.

Charging

Please NOTE that the Mini Crosser can be equipped with several types of charger (ask your dealer for information on the various types).



IMPORTANT! Only ever use a charger designed for charging dry maintenance-free batteries. The max. charging current is 12 A.

If charging is to take place outdoors, an enclosed charger without a fan should be chosen.

The battery indicator shows how much power is available to the Mini Crosser.

- Red, yellow and green indicate that the batteries are fully charged.
 - Red and yellow indicate that the batteries will soon need recharging.
 - Red indicates that the batteries need to be recharged as soon as possible, otherwise the Mini Crosser will cut out.
-

The battery manufacturer recommends that the batteries are charged at a temperature of + 10° to +30° C in order to achieve a charging time as specified in the section Technical data.

The battery capacity is approx. half at -10 ° C compared to the capacity at + 20 ° C. It is recommended that the batteries be charged every night if the Mini Crosser has been used.

Avoid deep discharge. We recommend charging in a heated room. If this is not possible every day, it is a good idea to charge the Mini Crosser in a warm (approx. 20 °) room once a week.

New batteries only reach full capacity after approx. 20 times discharges and charges.



NOTE! that the capacity of the batteries decreases with time and at low temperatures. In practice, this means that a vehicle with old batteries has less mileage than it had when the batteries were new.

When driving in hilly terrain, on slopes and up curbs, the distance that can be driven before recharging the battery is reduced.

The Mini Crosser should charge when the vehicle is not in use. The type of charger supplied by the factory automatically switches to maintenance charging (very low power consumption) when the batteries are fully charged. Therefore, leave the charger connected until the Mini Crosser is to be used again. The charger cannot overcharge the batteries.



JK Medico charger

Important!

Don't leave the charger on the seat during charging.

The charger will flash until it is finished. Then the indicator on the charger will be constantly lit.

Connecting the charger



Connect the plug to the wall socket



Connect the charging plug to the 3-pin socket on the steering column. The charging socket is located under the protective cover.



Note

If the battery is 100% discharged (deep discharge), it is not possible for the charger to start charging the battery.

There is no warranty on batteries that have been damaged as a result of deep discharging.

If you are not using the Mini Crosser for a longer period of time, for example during winter storage, you can choose to disconnect the one pole to the fully charged batteries. This way there will be no consumption of standby power.

Battery disposal

Used batteries must be disposed of through your supplier or at a recycling centre.

Take care when handling any leaky batteries, as they contain corrosive acid.

INFO! New batteries can be purchased from Medema A/S.

Fuel for heater

The tank for fuel is placed behind the seat, and filled from the outside.

Use **ONLY** paraffin or diesel.




In the front, right side of the Mini Crosser X Cabin, you find the the exhaustion from the heater.



NOTE! It gets **VERY** hot.

Exhaustion from heater.

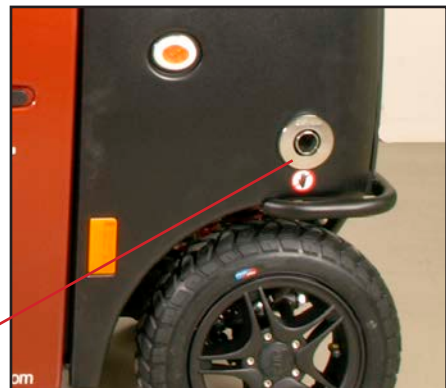

Heater



Heating button:


- On/off-button / Reset error
- Combustion chamber ventilation
- Menu confirmation

Heating regulator:

- Power/temperature settings
- Menu selection

Ventilator:

- Ventilator on/off button
- Cancel timer
- Confirm menu
- Exit

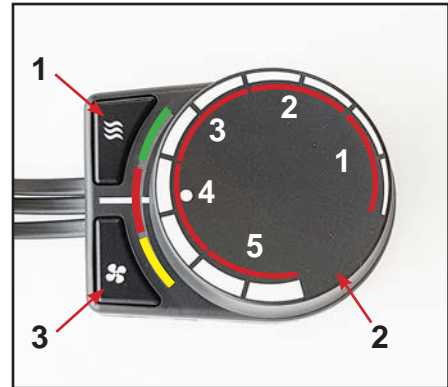

Automatic Heating:

- A Long 2 sec push on button 1 will activate automatic heating mode. If the temperature measured internal in the control unit is above the temperature setpoint of the knob the Yellow LED will be on. When the measured temperature falls below the setpoint temperature the heater will enter heating mode.
- When heating mode is activated the Green LED will flash during starting up. Once normal operation mode is reached, the green LED will be constantly on. When the temperature setpoint is reached the heater will shut down. While the heater is shutting down the green LED will flash. After shut down the yellow LED will be on indication the heater is still in automatic heating mode.
- A long 2 sec push on button 1 will deactivate automatic heating mode and activate idle mode.



- Button 2. The knob adjusts the temperature setpoint between 0 °C and 30 °C by turning it.

Continued on next page



Power setting and Error reset button

- If the heater is in error mode then use button 3 to reset error.
- If the heater is not in error mode button 3 can be used to adjust the power output of the heater. There are 5 power settings corresponding to 5 fuel rates: 0.12, 0.14, 0.16, 0.18, 0.20 l/hr. To adjust the power setting follow below sequence:
 1. Push button 3. This disables the thermostat. Yellow LED is on.
 2. Turn the knob to the desired power setting. Yellow LED is on.
 3. Push button 3. This selects the power setting. Yellow LED is flashing.
 4. Turn the knob back to temperature setpoint. Yellow LED is flashing.
 5. Push button 3. Enables the thermostat. Yellow LED is off.

LED Green

- Constantly lit when heater is heating and startup sequence is finished.
- Flashing more on than off indicating the heater is in starting up mode.
- Flashing more off than on indicating the heater is in shutting down mode.

LED Yellow

The LED is constantly lit.

- Heater is in automatic heating mode waiting for the room temperature to fall below setpoint temperature.
- Or heater is in power setting mode.

Flashing:

- The heater is in power setting mode.
- Or heater is signalling an Error together with the red LED

LED Red (signaling)

- Error code with 0.4 Hz frequency and with 8-second gap

No LED signaling:

- Heater is in idle mode

Note!

If the heater does not start after two attempts when I press the on/off button, it is necessary to open the front door and remove the fuse, then reinsert it.



Fuse for heater



IMPORTANT!

The heating ribs by the window and by the legs should NOT be covered up when the heating is on. That will cause overheating of the heater



Windscreen wiper / washer

Start the windscreen wiper by turning this knob.

Notice the smooth speed adjusting.

*Windscreen wiper - turn the knob.
Speed: up=faster - down=slower*

Washer - press this button



Signal condition


The three LED indicators show what you have set the wiper menu to. By pressing one of the buttons, the LED indicator lights up:

- flashes as many times as the delay time (in seconds) between wiper strokes is specified. (If the delay time between wiper strokes is 4 seconds, the LED flashes 4 times)

Mode	LED (■=on)	Signal
Continuous speed	■ ■ □	A little wink
2 sec delay	□ □ ■	2 slow flashes
4 sec delay	□ ■ □	4 slow flashes
8 sec delay	■ □ □	8 slow flashes
Off	□ □ □	LED OFF
Error code	Asymmetric.	Continuous


The washer fluid reservoir is located under the front cover.

Replacing the wiper blade

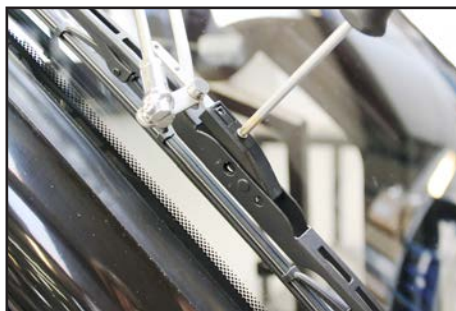

To exchange the wiper blade, do the following:


Screw this screw out.

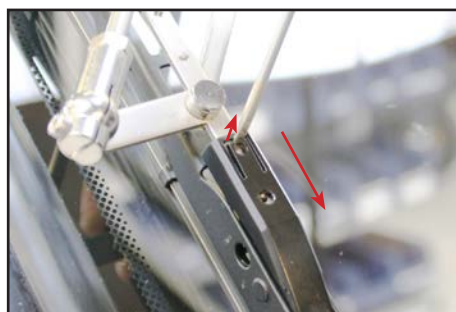

Lift carefully, so you can pull the wiper blade over the pin.

Mount a new in reverse order.

Replacing the headlight

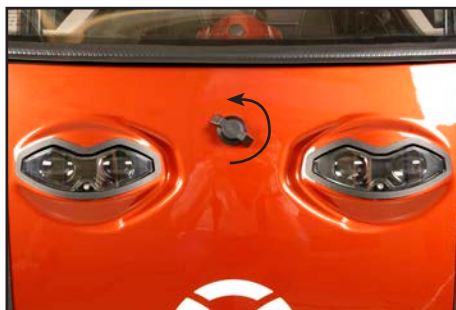

Open the front cover, by turning the handle 90° counter clockwise, and pull.



The headlight is attached with a fitting on the back of the screen. A single finger screw holds it firmly in place. Remove the screw.



Remove the headlight. Detach the plug, and it is free to change.



New headlight set.

Mount in reverse order.



The headlight can be adjusted up/down by turning this screw.



NOTE!

Be aware that drain lists are not trapped when the front cover is closed.


It **MUST** cover the edge of the front cover.

Changing the wheels

If you get a puncture in one of the pneumatic tyres or if a tyre is so badly worn that it needs to be replaced, follow the instructions below.

Tyres and inner tubes can be purchased from the authorised dealer who supplied the Mini Crosser.

The Mini Crosser Xust be **TURNED OFF** before you start.

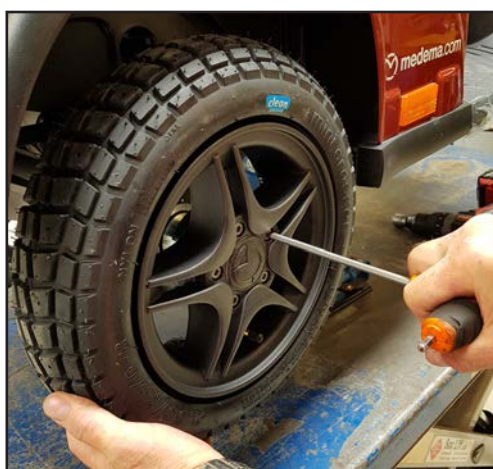

Changing the wheels on the 4W model

Remove the hubcap.
5 mm Allen key

- Undo the five bolts.
- Take the wheel off.
- When the wheel is put back on, the bolts must be done up tightly.



Remove the valve cap and use a screwdriver or similar to open the valve so that the air can be released.



REMEMBER!

- to let all the air out before taking the wheel apart to repair a puncture



Important!

There is a risk of explosion of the wheel, if the two rims are being taken apart with air in the tire.



Remove the flange with the 5 screws (5 mm Allen key).



Replace or patch the tube. Ensure that there are no foreign objects in the tyre before the tube is replaced. Pump air into the tube so that it is positioned correctly in the tyre, but do not fill it up completely yet.


In this way the tube will not get jammed, when the flange is mounted again.



Put the flange back on.

Pump up the tyre to the correct pressure. (See technical data)

Put the wheel onto the Mini Crosser again.

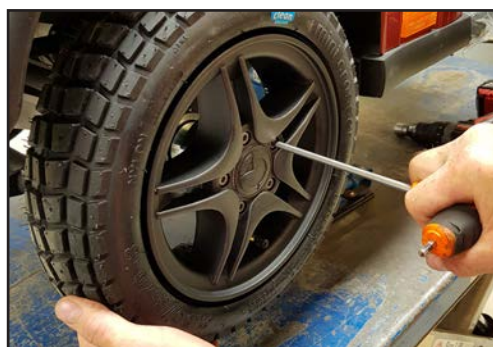


Note!

The spring washer is part of the original screws.

Fuses

The Mini Crosser X1 Cabin has the following protection systems:


- Automatic overload protection, which limits the current to the motor. This protection is integral to the control system and cannot be adjusted. It prevents both the control system and the motor from being overloaded. In the event of overload the current to the motor is reduced - full speed and traction can be resumed after the motor has cooled down for 2-5 minutes.
- The control fuse protects against mains faults. It is located under the rear cover.
- The main fuse protects against short-circuits in the main current circuit. The fuses are located under the rear cover behind the batteries. Fuse type DIN 2581.
- If the ignition key is turned at the same time as the accelerator is operated, the vehicle will not move off for safety reasons.
- The lights and horn are protected by electronic fuses, which protect against any short-circuiting that may occur. To prevent accidental short-circuiting when a bulb is changed, the Mini Crosser X1 must be turned off when the new bulb is fitted and then turned on again afterwards.

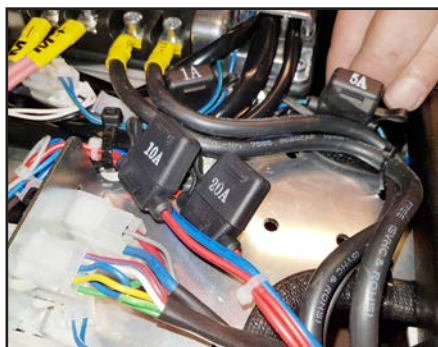

Take the rear cover off. The plastic cover can be removed by squeezing the sides. Now the fuses can be replaced.



Open the front cover, by turning the handle 90° counter clockwise, and pull.

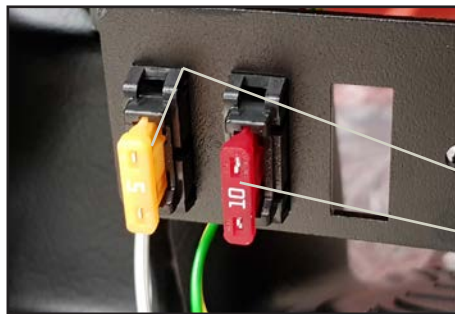

Under the front cover you will find the fuse for the heater and windscreen wiper.

Fuse for windscreen wiper.

Fuse for the heater and the control panel



NOTE!

Be aware that drain lists are not trapped when the front cover is closed.

It **MUST** cover the edge of the front cover.

Brakes

There are four braking systems on the Mini Crosser:

- Motor brake - adjusts vehicle speed also when going downhill.
- Magnetic brake - the magnetic brake is automatic and engages when the Mini Crosser stops. In an emergency, the Mini Crosser can be stopped instantaneously by turning the key switch. Please note that this will cause very sharp braking. The rear wheels will lock.

Must NOT be used in the normal course of driving.

The brake must never be disengaged mechanically using the disengagement lever on a slope. This function is only designed for use when pushing the Mini Crosser on a flat road.

- Electric safety brake - if the brake is somehow disengaged on a slope/hill, the Mini Crosser will brake automatically when it reaches a certain speed. There is an electric safety feature in the control system, which works even if the battery is disconnected. This is also the reason why the Mini Crosser cannot be towed at more than 5 km/h. See the section about towing.
- Hand brake - intended as an emergency brake and parking brake. It must be operated with caution when driving in slippery conditions and downhill.

Warning!



If there is a fault in the gearbox, the scooter can be stopped by using the mechanical handbrake. Since the brake is on the left rear wheel, turning left and braking at the same time can lead to loss of grip and braking effect on the left wheel.



When using the hand brake as a parking brake, lock it in braking position by pressing the button in while applying the brake.

To release the brake, press the button again.

Hand brake lock

Disengagement

Manual

The manual disengagement lever is placed underneath the back cover. Follow this procedure to use it.

- Turn the key to (0) position.
- Push the disengagement lever down.
The motor brake is now disengaged and the vehicle can be pushed or towed, but not drive itself.
- Push the disengagement lever up
Now it can run normally. It can no longer be pushed or towed.




NOTE!

Don't ever disengage on sloping terrain.

If the brake is somehow disengaged on a slope/hill, the Mini Crosser will brake automatically when it reaches a certain speed. There is an electric safety feature in the control system, which works even if the battery is disconnected. This is also the reason why the Mini Crosser cannot be towed at more than 5 km/h. See the section about towing.

Once the motor brake has been disengaged, the Mini Crosser can only be stopped with the hand brake.

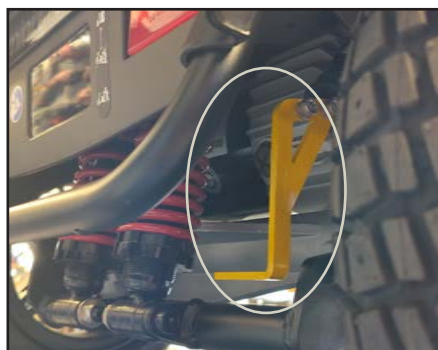

Push the disengagement lever down to disengage (only hand brake works).



And push up to reactivate the motor (normal driving now possible).



Securing to vehicle floor with belts



Dahl Engineering belt set for securing in motor vehicles.

ALWAYS use four belts at the back and two at the front.

The belts must always be attached to approved fittings in the vehicle and the four eyes welded to the Mini Crosser.

The belts **MUST** be attached within the angles shown in the picture for optimum security.

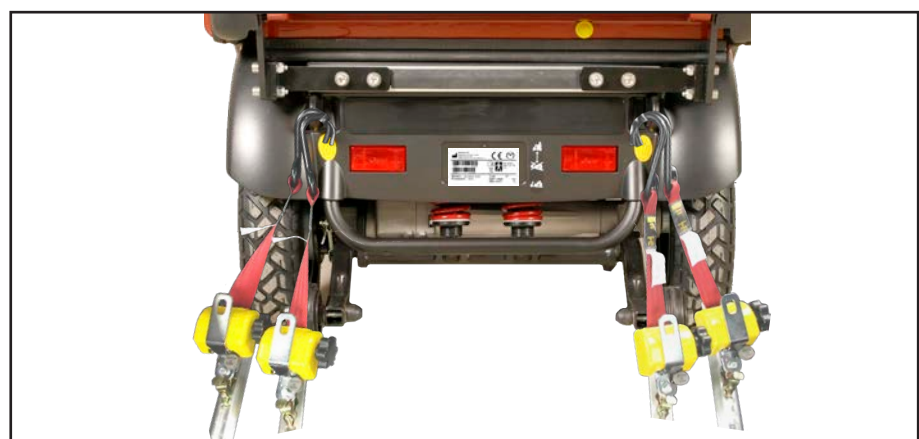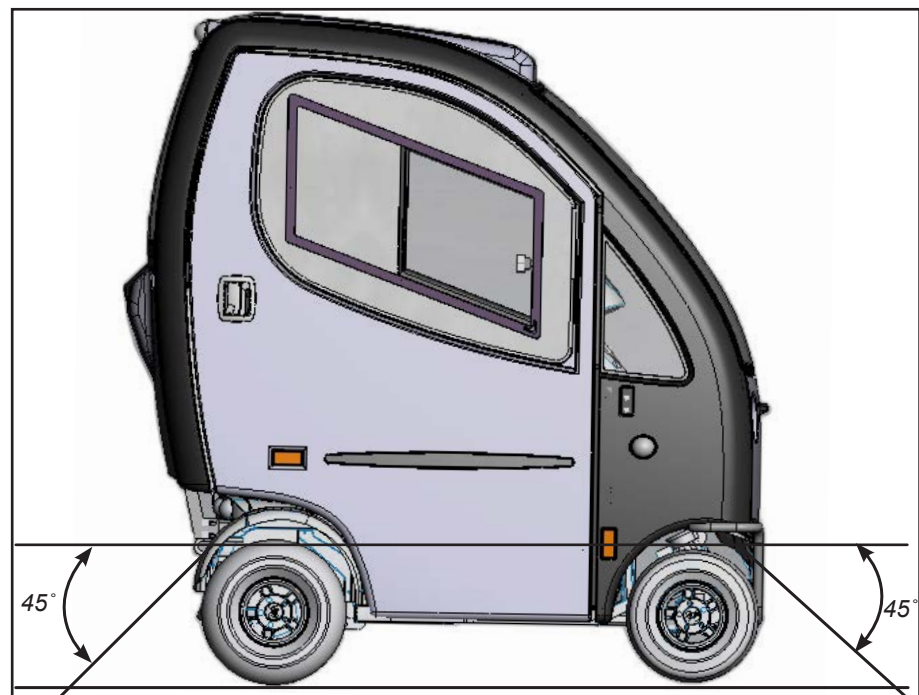

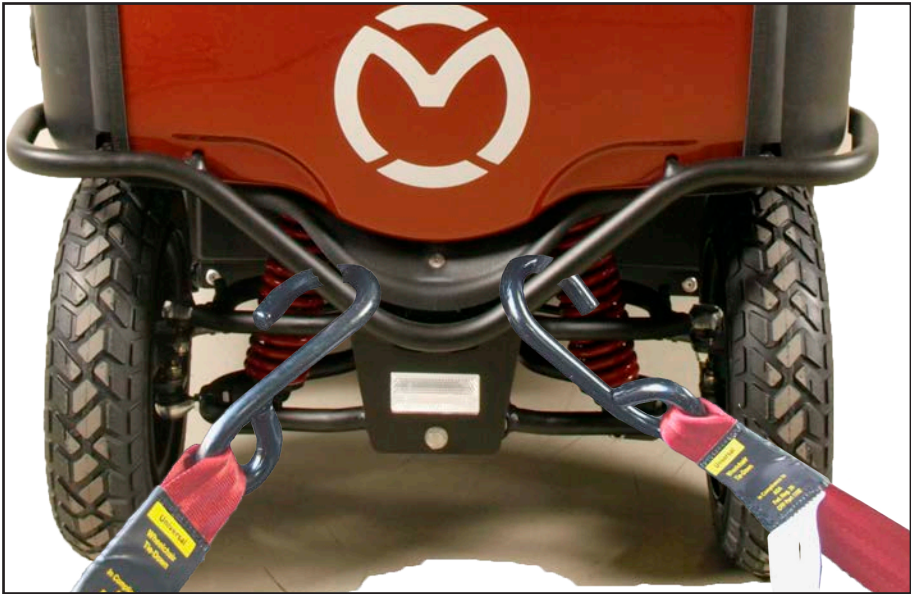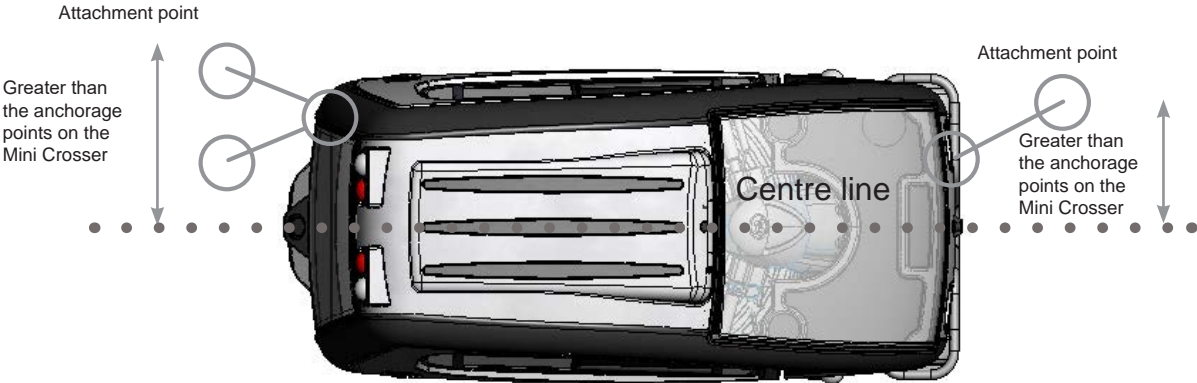

Transporting by plane

If the Mini Crosser is to be transported by plane, the airlines require:

- the batteries to be flight-approved
- the air to be let out of the tyres
- the battery leads to be disconnected (not always, but frequently)

The seat and cover have to be removed to disconnect the battery leads.



Lift the armrests up, and fold the seat completely. Release the seat turn - turn the seat a little and lift it of.



Remove the thumbscrew and remove the battery cover.



Disconnect the (+) pole.

A battery declaration for air travel can be requisitioned from Medema A/S. See contact informations on page two.

Towing



If you should be unfortunate enough to break down, the Mini Crosser can be towed or pushed. The Mini Crosser must always be turned off and the motor brake disengaged during towing. See the section on Brakes.



If the Mini Crosser is to be towed, secure a rope to the tow fitting on the front - marked with a yellow "hook mark". Do not tow faster than 5 km/h. The Mini Crosser will generate electricity when it is towed, with the motor acting as a dynamo. If it is towed at more than 5 km/h, there is a risk of the motor generating enough electricity to damage the Mini Crosser and, in the worst case, cause a fire.

The Mini Crosser will try to brake if it is towed at more than 5 km/h.

Dimensioned drawing, Mini Crosser X1 Cabin

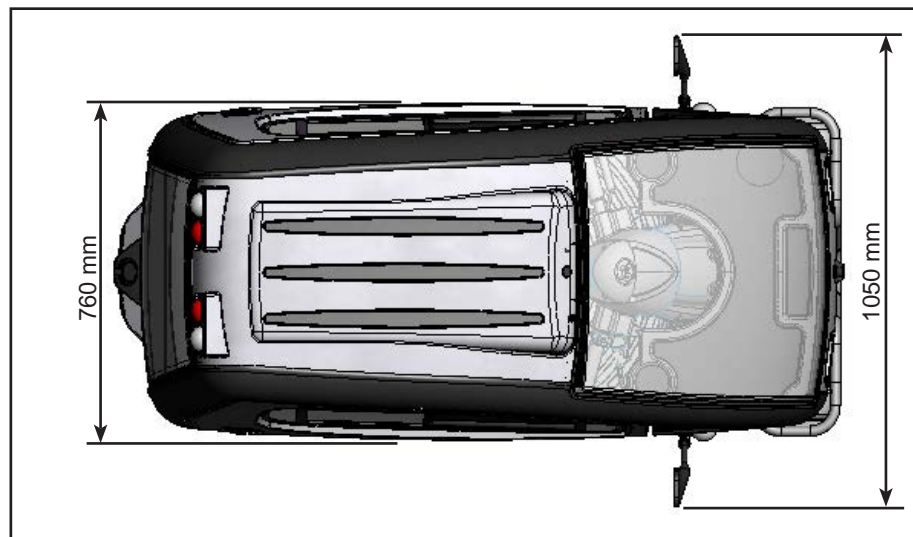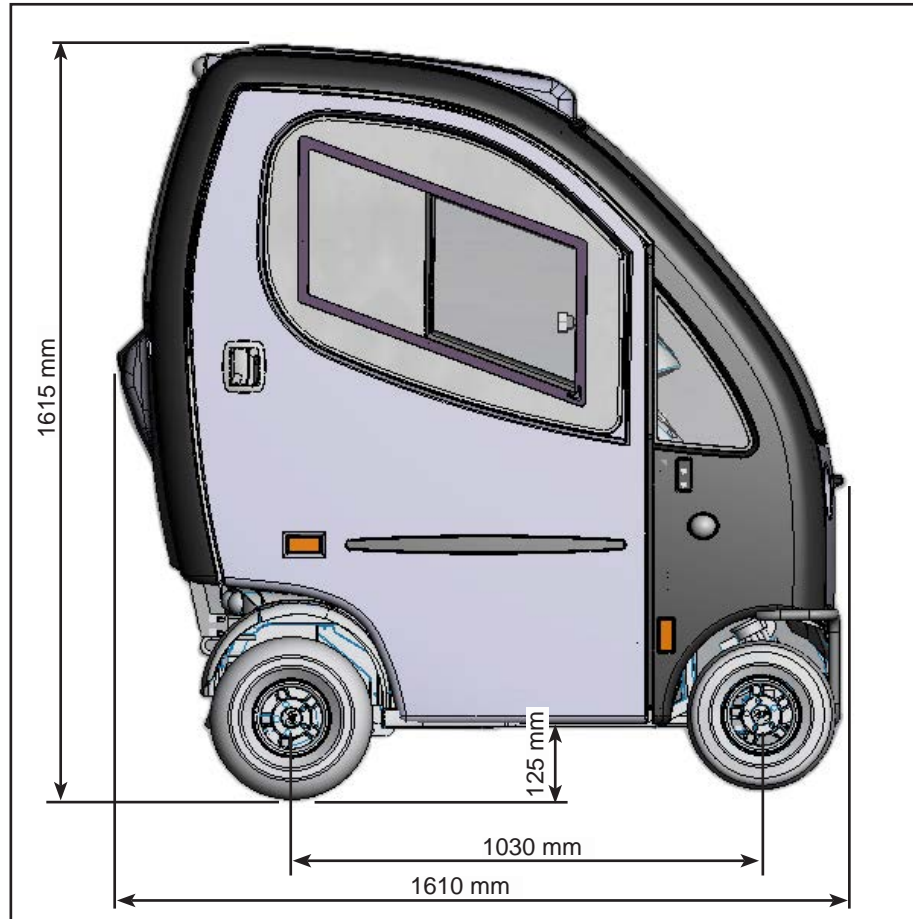

Introduction to the Ergo2 seat

The Ergo2 seat is designed to stay safe to use for at least 10 years.



Lever for rotating the seat

The lever must be pulled back to rotate the seat. The seat can then be rotated to each side, locking at each 45°.

The lever is spring-loaded, and the seat is locked automatically when the lever is released. This lever is located on the right side as standard, but can also be positioned on the left side. When the lever is located on the left-hand side, it must be pushed forward in order to rotate the seat instead of pulling back.



Out / in adjustment of the lever for rotating the seat

Lift the seat off the vehicle.

Loosen the two Allen screws and the lever can now be adjusted out and in to the desired position.



Placement of lever for rotating the seat on the left-hand side

Remove the two Allen keys and pull the lever out. Insert the lever from the left-hand side and tighten the Allen screws.

When installing the lever on the left-hand side, the lever must be pushed forward to rotate the seat..



Mount the seat again

In order to ensure that the seat returns to the correct position, the lever for rotating the seat must be released in order to place the seat.



Lever for fwd/bw adjustment of the seat.

Pull the lever up to release the seat on the slide rail.

Then the seat can be moved forward or backward as desired with 200 mm of travel.

When the lever is released, the seat will automatically lock into place in the closest position.



Height adjustment of armrest

Unzip the zipper on the side of the backrest and the backrest itself.



Unscrew the Allen screw and the armrest can be pushed up or down as needed. It can be adjusted 140 mm.



Width adjustment of armrest

Each armrest can be adjusted 25 mm on each side. Loosen the Allen screw and the armrest can be adjusted longitudinally.



Adjusting the angle of the armrests

The adjustment screw makes it possible for the armrest to be adjusted 60°.

The armrest can be tilted up to facilitate entry and exit.



Adjusting the angle of the backrest.

The angle of the backrest can be adjusted 45° backward and 90° forward. This is done by pulling the lever on the right-hand side of the seat.



Backrest tilted 45° backward.



Backrest tilted 90° forward.

This is why the seat does not necessarily need to be removed during transport, e.g. in a car.



Backrest forward / back.

Loosen the specified screws on both sides.



The depth of the seat can now be adjusted by sliding the seatback forward and back.

Shown here with 320 mm seat depth.

The seatback can then be moved backward to the edge of the rail and forward to the desired position. Tighten the screws after adjusting.



Shown here with 550 mm seat depth



Height adjustment of headrest.

Press the button and the headrest can be raised and lowered as needed.



Adjustment of the headrest forward and backward.

The headrest can be tilted forward and backward as needed.

Flame resistance

The Mini Crosser seat's flame resistance has been tested in accordance with DS/EN 1021-2:2014 Furniture - Assessment of the ignitability of upholstered furniture - Part 2: Ignition source match flame equivalent.

Warning!



The seat may become very hot if exposed to direct sunlight. Similarly, the seat will become very cold if it is exposed to cold temperatures, e.g., frost.

Basket on the back of the cabin



**Mounting of basket on the cabin.
(Optional accessory)**



The basket with the mounting bracket. You have to adjust the width by loosening the four screws - pull or push until the width fits.



Mount the basket on the brackets and now it is ready for use.

Remember to insert a locking split after mounting.



Troubleshooting

The following is a list of various problems that, in our experience, may occur. The list gives possible causes and remedies.

Problem	Possible causes	Remedy
The Mini Crosser will not go. The battery indicator is not lit.	The key has not been turned to start. The batteries are completely flat. The control fuse has blown. The main fuses have blown.	Turn the key and wait 5 sec. before activating the accelerator. Charge the batteries. Change the fuse. Contact supplier.
The Mini Crosser will not go, but the battery indicator is lit.	The Mini Crosser has been overloaded. The hand brake is on. There is a fault in the electronics. The batteries are flat. The charging plug has not been removed.	Wait approx. 1 min. before trying again. The vehicle must be turned off (see section on Fuses). Release the hand brake. Contact supplier. Contact supplier. Remove the charging plug.
Mini Crosser drives unevenly, choppy while driving.	Can be caused by poor connection of electricity.	Turn off the Mini Crosser immediately and seek out an authorised workshop. If you continue driving, heat can be generated in the poor connection resulting in a risk of fire.
The driving speed is too low.	The speed selector is on slow. The electronics are overloaded. There is too little air in the tyres.	Change to a faster speed. Stop and wait a few seconds before starting. Pump the tyres up to the right pressure.
The driving distance per charge is too short.	There is a problem with the batteries. Low temperature. There is a problem with the charger. There is too little air in the tyres. The charging method is wrong.	Charge the batteries and check that the green lamp on the charger lights up before driving off. Contact supplier. Pump the tyres up to the right pressure. Read the section on Charging in the User Manual.
The charging lamp on the charger does not light up when the charger is connected to the mains and the Mini Crosser.	No power to the switch. Fault in cable. Fault in charger.	Turn the switch on. Contact supplier. Read the operating instructions for the charger. Contact supplier.

Problem	Possible causes	Remedy
The "ready" lamp on the charger does not light up even though the charger has been on for 10-12 hours.	There has been a power cut. The charger is doing a top-up charge. There is a problem with the batteries. There is a fault in the charging plug for the Mini Crosser X1 Cabin.	Reconnect the charger and repeat the charging process. Check again half an hour later. Contact supplier. Push the charging plug all the way in and repeat the charging process. Read the operating instructions for the charger.
The "ready" lamp on the charger lights up even when partly discharged batteries are connected.	The fuse in the charger has blown. The switch in the charging plug is malfunctioning.	Contact supplier. Contact suppliers - read the operating instructions for the charger.
The charger lamp is showing an error.	The charging plug has not been inserted or there is a mains fault. The battery voltage is too low for charging to start.	Push the charger plug in or contact the supplier. Read the operating instructions for the charger - or contact the supplier.
The heater doesn't work.	Need paraffin/diesel. The fuse has blown	Put on paraffin og diesel. Change the fuse. Read the Webasto user guide for more information.
The washer doesn't work.	Washer fluid i missing. The fuse has blown.	Put on washer fluid. Change the fuse.
The windscreen wiper doesn't work.	The fuse has blown.	Change the fuse.

Electronic faults

If there is an electronic fault, a number of lamps in the battery indicator will light up. The following table shows what they mean.

Lamps	Causes	Remedy
1 lamp	The batteries need charging or the connection to a battery is poor.	Check all connections between the controller and batteries. If these are fine, try charging the batteries
2 lamps	Poor connection to the motor.	Check the connection between the motor and controller. Contact supplier.




Lamps	Causes	Remedy
3 lamps	Short-circuit between the motor and a battery connection.	Contact supplier - say how many lamps are lit.
4 lamps	Not in use	
5 lamps	Not in use	
6 lamps	<p>The 200 is prevented from driving. Inhibit 2 is active. This may be due to the battery charger being connected, the plug to the thermal switches being disconnected, a fault in the thermal switch or the Mini Crosser is overheating.</p> <p>The Mini Crosser will turn itself off automatically after it has been stationary for 10 minutes. The battery indicator flashes with 6 lamps at a rate of about 3 seconds.</p>	<p>Stop the unit and wait 3-5 minutes before restarting. Turn the key to 0 and then to 1 to reset the fault.</p> <p>Contact supplier.</p> <p>To start X1 again, the key must be turned to position 0 and back to 1.</p>
7 lamps	<p>Accelerator fault.</p> <p>Overheating</p>	<p>Make sure that the accelerator is in neutral when the Mini Crosser is turned on.</p> <p>In case of overheating, turn off the Mini Crosser and wait for it to cool down sufficiently, then turn it on again.</p>
8 lamps	Controller fault.	Contact supplier.
9 lamps	<p>Poor connection to motor brake.</p> <p>Magnetic brake is disengaged.</p>	<p>Check the connections from brake to controller, or push the brake lever up.</p> <p>Check that the vehicle is not disengaged.</p>
10 lamps	The controller has been supplied with a high voltage in excess of 38 V. Usually seen in the case of a poor connection.	Check all connections from the batteries to the controller.
Running lights	Charger is connected.	Remove charger.





Programming



NOTE! For safety reasons, modifications may only be made by trained personnel, such as Mini Crosser service engineers and consultants or authorised service personnel at mobility centres, etc.

Technical data

Technical data	X1 Cabin
General information:	
Transport height without seat (Steering column folded down)	161,5 cm
Total length:	161 cm
Total length incl. anti-tilt wheel:	161 cm
Total width:	76 cm
Total weight incl. batteries and Ergo Std seat.	243 kg
Weight without seat (Ergo Standard 45 cm).	217 kg
Dynamic stability in all directions.	10° - 17%
Kerb climbing	11 cm
Max. speed	
Option 1	10 km/t
Option 2	13 km/t
Option 3	15 km/t
Braking distance	
10 km/h	2,0 meter
13 km/h	2,8 meter
15 km/h	3,5 meter
Controller	S-200
KW transaxle	0,7 kW
Turning circle / radius	330 cm / 165 cm
3 point turning circle / radius	190 cm / 95 cm
Max user weight - standard	150 kg
Wheels	
1503-1016 Wheel, model-X, 2,50-3,30-8", coars, complete with rim and tire.	2,8 bars 
1503-1400 Wheel model-X 2,50-3,70-8", coarse, Outw. Ø326 mm w/spikes.	2,8 bars 
1503-1396 Wheel, model-X, 2,50-3,70-8", cooars, Complete puncture free PUR.	2,8 bars 

Technical data	X1 Cabin
1503-1402 Wheel, model-X, 2,50-3,30-8". complete with rim, tire, and armor insert.	2,8 bars 
1503-1003 Wheel, model-X, 2,50-3,30-8". complete with rim and tyre.	2,8 bar 
1503-1008 Wheel model-X 2,50-3,70-8" Outw. Ø326 mm w/ spikes	2,8 bar 
1503-1012 3,50-8" ext. Ø370 mm Wheel, model X Used only as rear wheels and never with 15 km transaxle	2,5 bar 
Classification, ISO	Class C
Seats:	
Effective seat width Ergo2 child Ergo2 standard	35 cm 40, 45, 50 cm
Effective seat depth Ergo2 child Ergo2 standard	20-43 cm 32-55 cm
Seat plane angle	3°
Backrest height Ergo2 child Ergo2 standard)	44 cm 54 cm
Seat surface height from ground to front edge Ergo2 (child, standard)	59 - 69 cm
Seat surface height from footrest to front edge Ergo2 (child, standard)	42 - 52 cm
Backrest angle Ergo2 (child, standard)	-90° til +48°
Batteries:	
90 Ah batteries. 10 km/h 13 km/h 15 km/h	(*) 35 km 38 km 40 km
Battery type: Standard:	2x12V / /90 Ah
Battery measure i cm. 90 Ah	26 x 16,8 x 21,1
Battery weight pr. stk. 90 Ah	26 kg

Technical data	X1 Cabin
Energy consumption in kWh, when charging from "empty". 90 Ah	Ca. 2,4 kWh
Charger, 24 V DC 90 Ah	8 - 12 A
Approx charging time at 20 ° C	8 hours
Lighting:	
Bulb, headlight	24 V - 18 W
Diode, rear light	24 V - 2 W
Diode, indicator	24 V - 2 W
Standard colour	Grey
Noise	
Noise level	60db
The Mini Crosser X1 Cabin complies with the following standards:	
The Mini Crosser has been tested for the requirements described in:	EN 12184
ISO 7176-15 Annex A	
a) requirements and test methods for static, impact and fatigue strengths (ISO7176-8)	Conforms
b) Power and control systems for electric wheelchairs, requirements and test methods. (ISO 7176-14)	Conforms
c) climatic test in accordance with ISO 7176-9	Conforms
Miscellaneous	
Rear basket, max load	15 kg
Wheel bolt tighten max:	
Dry	12 Nm
Greased with fat	6 Nm
Fuel tank	2,5 L
Sprinkler tank	1,2 L

(*) Driving distance is depending on: temperature, wind, terrain, tyre pressure and user weight.

Maximum driving distance with new batteries at +20° on flat, firm surface
Optimum battery capacity is reached after approx. 20 charges / discharges.

Cassation



The Mini Crosser or parts (fx. batteries) of the Mini Crosser can, for a fee, be taken to the nearest dealer for disposal in accordance with current environmental regulations.

Weight of the seats



This is our standard Ergo2 45 cm seat, the heaviest part of the X1 to remove. The label showing the seat's weight is located as shown above.

Seat model	Weight
Ergo2 seat 35 cm	20 kg
Ergo2 seat 40 cm	26 kg
Ergo2 seat 45 cm	27 kg
Ergo2 seat 50 cm	29 kg
Ergo2 seat 60 cm	36 kg
Ergo2 seat 70 cm	37 kg

Changes/adjustments

Changes/adjustments made to the Mini Crosser scooter and MC Electric wheelchairs must be carried out in accordance to MDR.

Original accessories for Mini Crosser or MC Electric wheelchairs, can be mounted on the main product and retain the CE marking.

Non-original or 3rd party accessories:

No changes may be made that affect the functionality or safety of the product. Accessories that are not original must be CE marked and have the same functionality as original accessories. Original attachment points and installation instructions must be used when installing alternative accessories. If these criteria are met, the product retains its original CE marking. We emphasize that, as a manufacturer, we do not take responsibility for the combination itself, it must be assessed and documented by the technically competent person who carries out the assembly.

The requirement for a technically competent person is that they have the professional competence to work with medical devices.

Non-original spare parts:

Original spare parts must be used for repairs in order for the product to retain its CE marking. Exceptions are screws and nuts that can be replaced with a component that has at least equivalent quality/performance to the original spare part.

Own notes



medemagroup