



medemagruppen



Headrest height adjustment

Push the button, and the headrest can be adjusted up/down.



Armrest height and width adjustment

Unzip the zipper on the side of the backrest and the backrest itself. Loosen the allen screw. Adjust the armrest up/down. At the same time the armrest width can be adjusted up to 25 mm in both sides.



Armrest angle

The adjustment screw makes it possible for the armrest to be adjusted 60°. The armrest can be tilted up to facilitate entry and exit.



Headrest angle

Headrest can be tilted forward/backward as needed.



Handlebar height

Adjust the handlebar up/down. Remove the rubber cap. Loosen the allen screw. Adjust up/down and tighten the allen screw again.



Steering column angle

The steering column can be adjusted forwards/backwards using the handle. Push the lever down and adjust the steering column closer to you, or further away for a better riding position.




Back rest and seat depth

Loosen the screws in the C-rail on both sides. Pull/push the back rest to the desired position. Tighten the screws again.



Seat forward/backward

Pull up the lever to release the lever. Adjust the seat forw./backw. When the lever is released, the seat will stop at the nearest stop.

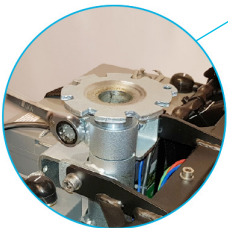

Seat height up/down

Remove the seat. Loosen the counter nut, adjust the seat post to the desired position. Tighten the screw and the counter nut.



Seat rotation

Pull the release lever back to rotate the seat. The seat rotates to either side with intervals of 45°.



Correct setting creates the right prerequisites

The Mini Crosser offers an advanced seat. A good sitting position is important to be able to ride the scooter safely and securely. We show here the most common adjustments to adapt the scooter to the user.





medemagruppen